



CIDEZYME™

ENZYMATIC DETERGENT
SOLUTION

MANUAL CLEANING

CIDEZYME™ Enzymatic Detergent Solution is a mild enzyme-based pre-soak and cleaner designed to effectively remove organic soil from contaminated endoscopic instruments and other medical devices prior to disinfection.



EXPERIENCE THE BENEFITS

CIDEZYME™ Enzymatic Detergent Solution is recommended to clean medical instruments before using the ASP High-level disinfection solutions.

- EFFECTIVE AND THOROUGH ACTION WITH PROTEASE ENZYME AND SURFACTANTS
- FAST-ACTING
- USER-FRIENDLY
- HIGH COMPATIBILITY WITH MATERIALS



CIDEZYME™

ENZYMATIC DETERGENT SOLUTION



→ FEATURES & BENEFITS

EFFECTIVE WITH PROTEASE ENZYME	that breaks down proteins like blood, mucous and feces contained in the biofilm with reduced mechanical action.
EFFECTIVE WITH SURFACTANTS	that improve the surface wetting of hard-to-reach areas like endoscope channels, and effectively clean away lipids and greases.
FAST-ACTING	begins to act in just one minute allowing fast turnaround of instruments.
EASY RINSING	its low foaming properties and neutral pH prevent it from leaving residues so that instruments can be effectively disinfected.
PLEASANT TO USE	with spearmint fragrance that masks the odors of dirty instrumentation.
EASY DISPENSING	with separate dosing pumps available.



MATERIALS COMPATIBILITY

CIDEZYME™ Enzymatic Detergent Solution can be used to clean a wide range of devices made from metals, plastics and elastomers, making it a very versatile cleaner that can be used anywhere medical instruments need to be thoroughly cleaned.

TECHNICAL INFORMATION

MEDICAL DEVICE CLASSIFICATION | Class I according to MDR (EU) 2017/745

IN-USE CONCENTRATION | Dispense 8 ml (0.8%) or 16 ml (1.6%) per liter of water

SOAK TIME | 1 to 3 minutes

USE LIFE | Use fresh as required

SHELF LIFE | 2 years

STORAGE | 15-30°C

DISPOSAL | Drain or as per hospital policy

ORDERING INFORMATION

REORDER NO.	DESCRIPTION	CASE CONTENTS
2258	CIDEZYME™ Enzymatic Detergent Solution	12 x 1 liters
2260	CIDEZYME™ Enzymatic Detergent Solution	4 x 5 liters

CIDEZYME™ Enzymatic Detergent Solution may also be used for cleaning instruments in automated equipment, in accordance with equipment manufacturers' recommendations.

FOR MORE INFORMATION, CONTACT YOUR LOCAL ADVANCED STERILIZATION PRODUCTS SALES REPRESENTATIVE

asp.com



ASP International GmbH, Zug Branch
Bahnhofstrasse 2, Zug 6300, Switzerland
©ASP 2021. All Rights Reserved.



ADVANCED STERILIZATION PRODUCTS, INC.
33 Technology Drive, Irvine CA 92618, USA



ASP, The Netherlands BV
BIC 1, 5657 BX Eindhoven, The Netherlands

